

Medical Ozone Generator AOT-MD-1000A



FEATURE

Patent Design

- ◆ **Designed in Germany, Key component Made in Germany**
- ◆ **Wide range Ozone concentration: 6.5-110ug/mL**
- ◆ **Built-in isolated ozone destructor for enhanced safety**
- ◆ **Long lifespan: Up to 5 years of reliable operation**
- ◆ **Continuous operation: Capable of running for 100 hours non-stop**


SPEC

Model NO.	AOT-MD-1000A
Housing Color	White
Oxygen Flow	1/16 L/min~ 4 L/min
Ozone Concentration	6.5 ~ 110 ug/ml (gamma)
Ozone Module	Corona discharge Titanium electrode with quartz tube
Ozone Destructor/Catalyst	Isolated
Input Voltage, Power	110-220V AC
Operating Temperature	25°C +/- 3°C
Storage Temperature	-0°C ~ 50°C
Product Size	300mm*145mm*85mm
Package Size	340mm*245mm*140mm
Net Weight	1.9KG

OZONE CONCENTRATION CHART


	4	3	2	3/2	1	3/4	1/2	1/4	1/8	1/16
M1 µg/ml	6.5	8.5	12.0	15.4	21.5	29.5	36.5	57	81.7	96
M2 µg/ml	12.8	16.8	23.0	28.8	39.2	51.6	61.8	86.4	104	110

DIAGRAM

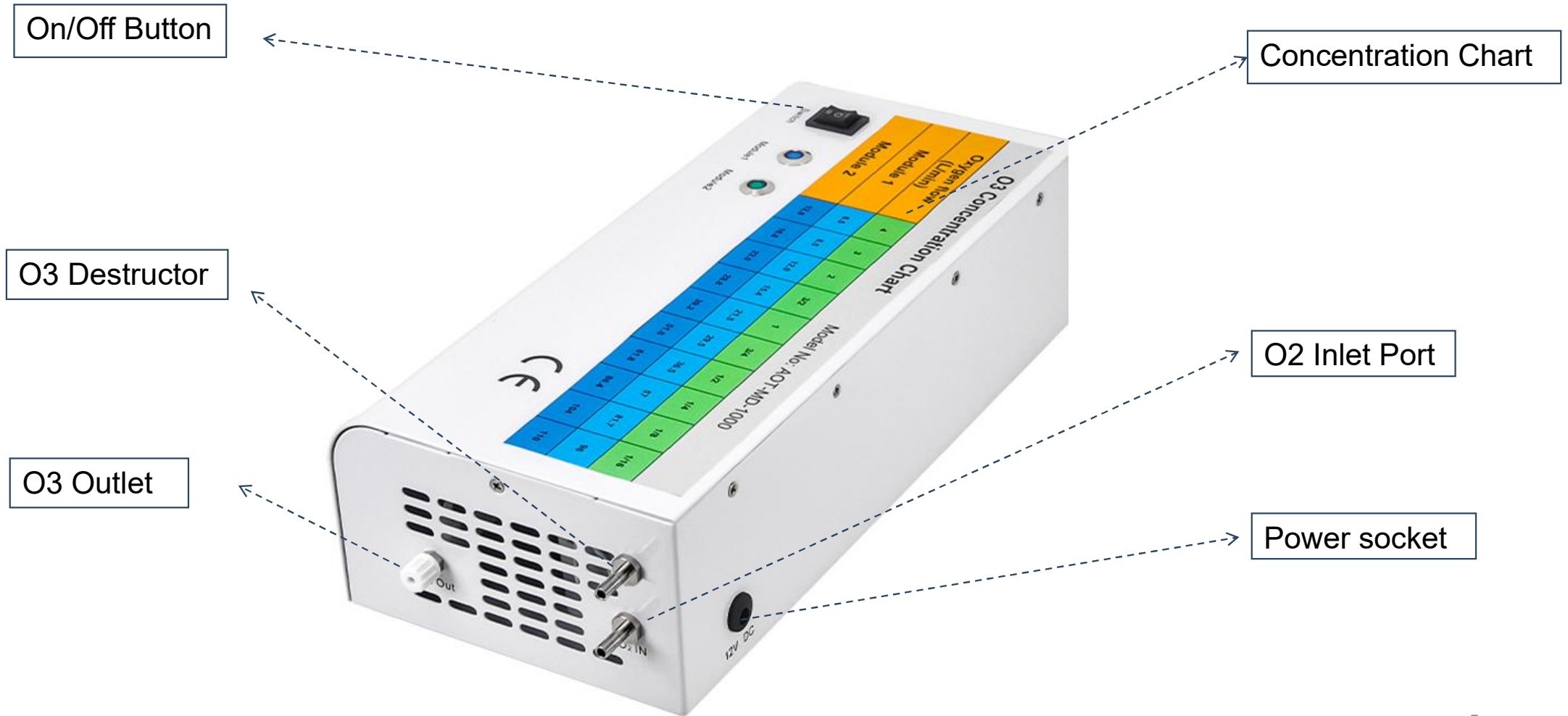


OPERATION


please make sure to operate the medical ozone generator with pure oxygen gas

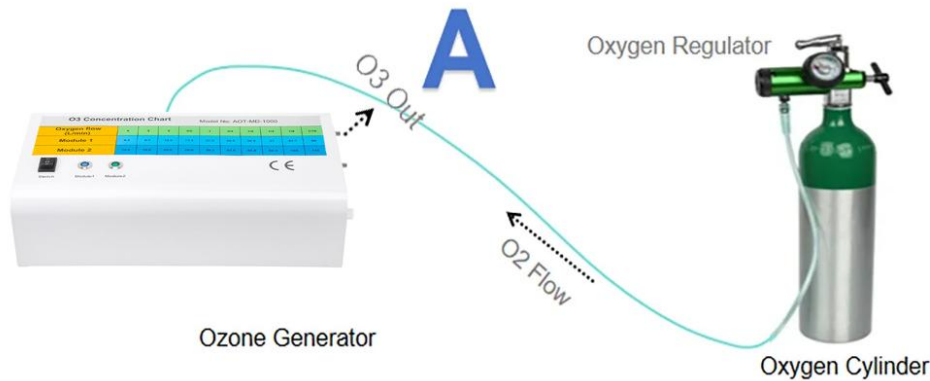

DO NOT ADD WATER to the oxygen concentrator! The ozone machine needs pure, dry oxygen

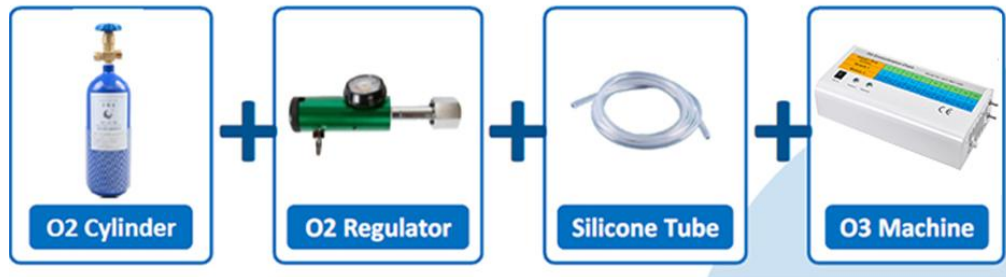

Connect with Oxygen Cylinder

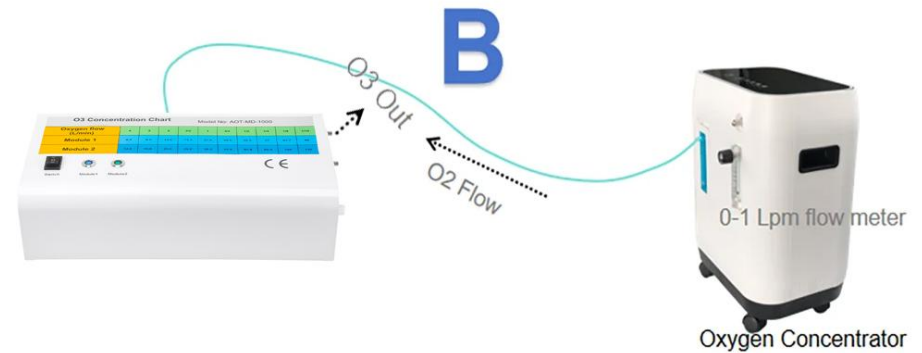

Connect with Oxygen concentrator

Package



34*24*14CM

medical ozone generator		1 piece
Male Luer Lock		2 pieces
3-valve lock		1 piece
Silicon house		2 meters
Stone diffuser		2 pieces
Adaptor 100-240V		1 piece

APPLICATION



Ozone Oil




intraarticular injection



Rectal Insufflation



Lymes Disease

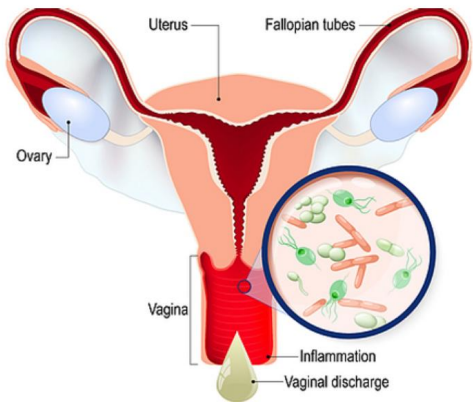


Gynecologic inflammation



Ozone Autohemotherapy



Ozone Sauna

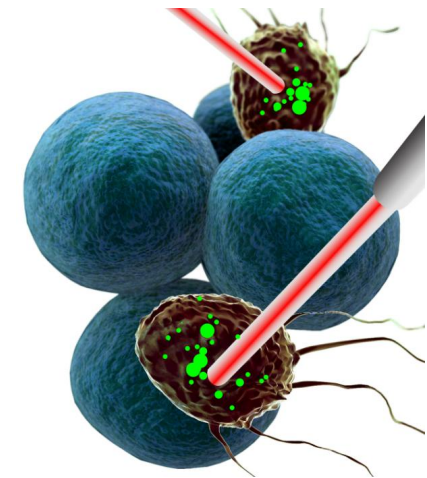

Cancer Treatment

THERAPY REFERENCE DOSAGE

POS.	OZONE APPLICATIONS	OZONE CONCENTRATION (1 ug/mL = 1 Gamma)	TREATMENT TIME /OZONE AMOUNT	FREQUENCY
1	Rectal Insufflation	30 - 45 ug/ml	125 CC ozone for beginners increase to 250 cc ozone slowly	<1 time /Day 1-5 times /Week
2	Bagging for Wound Healing	60-90 ug/ml to sterilize the wound & drop to 30-40 ug/ml to heal it	10-30 minutes	
3	Vaginal Insufflation	25-30 ug/ml	5-15 minutes per treatment	<1 time /Day 2-3 times /Week
4	Ear Insufflation	15-30 ug/ml	2-5 minutes per treatment	<1 time /Day 2-3 times /Week
5	Ozone Sauna	30-40ug/ml	10-30 minutes per treatment	<1 time /Day
6	Body Suit	25-30 ug/ml	15 minutes for beginners increase to 15-40 minutes slowly	<1 time /Day 2-3 times /Week
7	Intra Articular therapy	10-30 ug/ml	2 CC at 10 ug/mL for beginners increase to 10 CC and higher concentration slowly	2 times /Week
8	Ozone drinking water	20-80 ug/ml	0.5L water: 5—10 mins bubbling 1L water: 15-20 mins bubbling beginners start from low concentration	2 L /Day
9	Major Autohemotherapy	78 ug/ml	125-200 cc ozone for 125-200 cc blood	
<p>NOTE: 1. For Bagging or Body Suit, the ozone must be humidified (run through water) first 2. For Vaginal Insufflation, it should not be close to, or during the time of menstruation</p>				


OPTIONAL ACCESSORIES I



14Fr luer Catheter



Luer Syringe 30ml-200ml

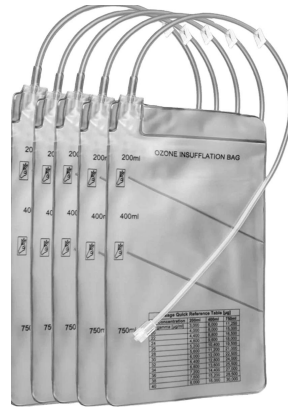

Rectal insufflation bag
Volume: 200-400-750ml



Ozone stethoscope
for ear insufflation



Ozone bag for half leg



Ozone bag for half leg



Ozone bag for half arm



Ozone bag for whole arm

OPTIONAL ACCESSORIES II



Ozone Bottle



Dental Hand Piece



Ozone Cap



machine suitcase



Vaginal Insufflation Kit



Ozone Sauna Kit

OPTIONAL ACCESSORIES III



AOT-OR-870
oxygen regulator



Bull nose
oxygen regulator



CGA 540
oxygen regulator
(thread 3/4")



CGA870



CGA540



Bull nose



ABOUT AQUAPURE

Aquapure is from Germany and was invested by Kenneth George Smith since 2002.

We are mainly manufacture ozone generators,plasma generators and anion generators in China. We Committed to the standardization of the ozone market, Our group is making great effort in the high quality and stable performance. Aquapure has more than 10 years' experience in the production and development of ozone generator.



Our Factory



Test Center



Contact Us

Headquarter: Aquapure Group GmbH

Hans-Thoma-Weg 3, DE 71032 Boeblingen, Germany

Telephone: 0049 7031 229688

Email: kgs-aquapure@gmx.eu

Aquapure (Shenzhen) Ozone Technology Co., LTD

Add: F5, Building #8, Jiuxiangling Industrial Park, Xili, Nanshan, Shenzhen, China -518055

Telephone: +86-18718054616

www.aquapureozone.com

NEW STICKER



PREVIOUS STICKER

